# Baby

italy • english • deutsch

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

CE 1011

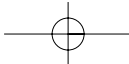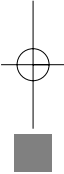**Safimpex**

## **ASPIRATORE NASALE SOFFY**

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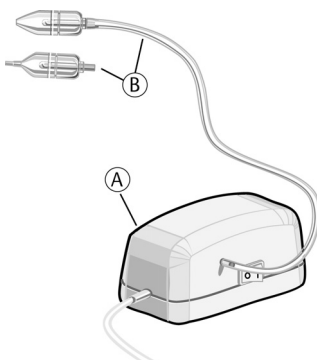
Ditta venditrice:

### **PARTS OF NANNY NOSE CLEANING DEVICE:**

- A** vacuum pump
- B** nose probes with silicon tube 2 pc  
instruction manual, guarantee certificate  
adapter (European or American)

### **IMPORTANT!**

Do not operate the vacuum pump if the external plastic casing or cord is cracked or broken.

### **FIELDS OF APPLICATION OF THE DEVICE:**

Infants and young children are unable to blow their noses by themselves. With the help of the nose cleaning device you can keep the baby's and young child's nose thoroughly clear.

Using the device does not substitute doctors' medical assistance.

The device can also be operated with two 1.5 V AA batteries, which can be inserted by removing the cover at the bottom of the casing.

Used batteries are environmentally harmful. Only dispose of them at designated collection sites, do not throw away.

### **USING THE NOSE CLEANING DEVICE:**

We distribute the nose-cleaning device with two types of probe head.

- We recommend the probe head with the cone end for general use. With this device the nasal cavity can be cleaned thoroughly. Sit the baby up in your lap. Place the nose-cleaning probe in one of the child's nostrils. The join of the nostril and probe should be airtight and you should block the other nostril with your finger. If you place the probe into the nostril correctly, mucus should appear in the probe within a few seconds. You should perform this operation on both nostrils.
- The nose probe with the silicon tube with a collar is for the suction of visible (fluid or clotted) mucus. Sit the baby up in your lap. Place the nose cleaning probe in the nostril in such a way that the end of the short silicon tube with the collar should reach into the mucus. The mucus will appear in the probe within a few seconds. You should perform this operation on both nostrils.

**WARNING:** The probe with the silicon tube with the collar must not be pushed all the way into the nostril, because it may cause discomfort to the baby, or might injure the nostril.

After successful nose cleaning you should cut the current off by turning it off, then unplugging the adapter from the socket. Pull the silicon tube attached to the pump off the suction stub. In case of fault or abnormal operation turn device off and unplug adapter from mains.

### **CLEANING THE NOSE PROBE:**

Pull silicone tube off the nose probe. Wash both ends of the probe in a detergent solution, rinse thoroughly, then sterilise by boiling for 30 minutes.

## ENGLISH

### **CLEANING THE VACUUM PUMP:**

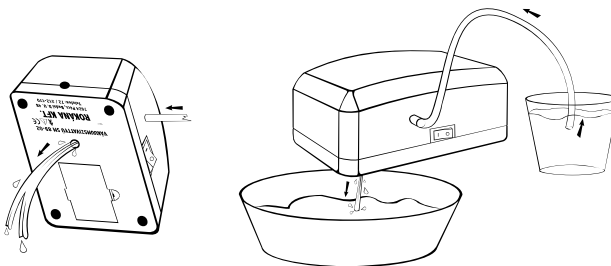
The outer casing of the vacuum pump should be cleaned with a cloth dampened with a detergent solution.

Should milk inadvertently get into the pump, it should be washed thoroughly as seen in the illustration.

#### ORDER OF CLEANING:

1. Fill a glass with lukewarm water.
2. Place the silicone suction tube of the pump into the water
3. Hold the device above a bowl.
4. Turn on the device.

It may be necessary to repeat this operation more than once.

Number of MEEI qualification: E0495E113, HU-344  
 ORKI qualification number: 393/1992  
 Adequacy marking: 

B type patient part  
 Class of protection against indirect contact: II

**TECHNICAL DESCRIPTION:**

Type: SM 89-02  
 Size of vacuum pump: 115 x 60 x 60 mm  
 Weight: 0,15 kg  
 Nenndaten:  
 Voltage:  
 U1 230 V ~50 Hz Power consumption: 7 W  
 U2 120 V ~60 Hz Power consumption: 6,5 W  
 Power supply unit: AC 230 V/DC 3 V 500 mA  
 AC 120 V/DC 3 V 500 mA  
 Power supply unit made by: FLJIKON  
 Model: 035-03-500

Continuous operation with intermittent duty  
 Loading time: 3 minutes,  
 Spacing time: 1 minute  
 Degree of protection: IP20  
 Vacuum: -0.4 bar  
 Application temperature: between 0 and 40 °C  
 Storage temperature: between 0 and 40 °C

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**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

**GUARANTEE**

Error indication:  
 Cause of error:  
 Fault clearing: Service

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