Baby

Elektrische Milchpumpe

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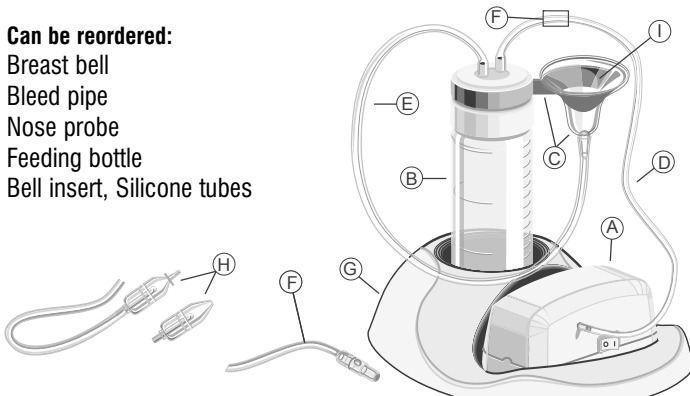
Verkaufsfirma:

ACCESSORIES OF NANNY MILK-PUMP

- A** vacuum pump
 - B** feeding bottle
 - C** breast bell with plastic holder
 - D** shorter (450 mm) silicone tube
 - E** longer (550 mm) silicone tube
 - F** silicon tube with bleed pipe (100 mm)
 - G** feeding bottle and vacuum pump holder
 - H** nose probes with silicone tube (2 pc)
 - I** silicone bell insert
- instruction manual and guarantee certificate
mains adapters (European or American)

Can be reordered:

- Breast bell
- Bleed pipe
- Nose probe
- Feeding bottle
- Bell insert, Silicone tubes

ASSEMBLY:

- Place the insert with the double outlet and the sealing ring into the lid of the feeding bottle.
- Screw the assembled lid onto the feeding bottle.

- Attach one end of the 550 mm tube onto the breast bell and the other onto the outlet protruding from the lid of the feeding bottle.
- Attach one end of the 450 mm silicone tube to the free outlet of the feeding bottle, and the other, with the 100 mm silicone tube with the bleed pipe inserted, to the vacuum pump.

IMPORTANT!

- Before first using the parts which come into contact with milk, they should be sterilised by boiling for 30 minutes.
- The vacuum pump must not be attached to the mains if a crack or damage is visible on the casing or lead of the device
- Using a damaged breast bell might cause accidents.

USING THE DEVICE:

Milk remaining in the breasts after breast-feeding causes inflammation, and will eventually lead to the drying up of the milk. This is why it is necessary to remove the remaining milk from the breasts. With the assistance of the NANNY (SM-89-02) electric milk pump the removal of the remaining milk can be performed comfortably, quickly and painlessly at home. The device is a closed system, so the milk reaches the bottle hygienically, and can be used immediately, or can be stored until the next feeding. One device for two tasks!

By attaching a special nose-cleaning probe to the NANNY vacuum pump a further function can be performed.

With its help uncongested nasal breathing can be ensured even in the season of nose colds, many illnesses can be avoided and the feeding of babies can be made easier.

The device operates with two 1.5V AA batteries, which can be inserted by removing the cover on the bottom of the casing. Used batteries are harmful to the environment, so they should only be disposed of at specially designated collecting points. They should not be thrown away.

ENGLISH**USING THE MILK PUMP:**

- Place the device and the feeding bottle in the holder and put it in a place where it cannot tip over and can be reached easily.
- Join the adapter to the mains (230V or 120V), and then to the vacuum pump.
- Place the breast bell on the breast, and then close the hole of the bleed pipe by holding the bleed pipe between the vacuum pump and the feeding bottle between the thumb and the index finger. After a short while milk will appear in the feeding bottle. By turning the bleed pipe between the thumb and index finger and quickly turning it back, the vacuum can be regulated. If you release the bleed pipe the vacuum will cease and the bell will fall off the breast. The other option is you remove the bleed pipe together with the 100 mm silicone tube and directly attach the 450 mm tube to the pump and to the feeding bottle. In this case the bell will cling to the breast and both hands will remain free to massage the breast. In this case the vacuum can be regulated by either pressing the breast next to the bell, or by moving the bell to allow air into it.
- The feeding bottle can be filled to a maximum two thirds capacity.
- After use turn off the device and detach the adapter from the mains.

CLEANING AND STERILISING THE DEVICE:

- Wipe the vacuum pump with a cloth slightly dampened with diluted detergent. Make sure no water gets into the device. The other parts (feeding bottle, sealing ring, insert with double outlet, breast bell, silicone tube coming into contact with milk) should be washed with ample solution of warm water and detergent, rinsed, then boiled for 30 minutes to sterilise and allowed to drip dry.
- The breast bell holder should not be boiled as it might become deformed.

IMPORTANT!

If the sealing ring is missing from the lid of the feeding bottle there is no vacuum, hence no suction. The same occurs if the lid of the feeding bottle is not screwed onto the bottle. The feeding bottle must be vertical during operation.

WHAT TO DO IN THE CASE OF MALFUNCTION

If there is no suction

- check the lid is properly closed
- check the silicone tube connections are properly sealed.

If the vacuum pump does not work check there is voltage in the wall socket. If a fault or malfunction occurs turn off the device and detach the adapter from the mains. Repairs to the vacuum pump can only be performed by the manufacturer.

USING THE NOSE CLEANING DEVICE:

We distribute the nose-cleaning device with two types of probe heads.

- We recommend the probe head with the cone end for general use. With this device the nasal cavity can be cleaned thoroughly. Sit the baby up in your lap. Place the nose-cleaning probe in one of the child's nostrils. The join of the nostril and probe should be airtight and you should block the other nostril with your finger. If you place the probe into the nostril correctly, mucus should appear in the probe within a few seconds. You should perform this operation on both nostrils.
- The nose probe with the silicon tube with a collar is for the suction of visible (fluid or clotted) mucus. Sit the baby up in your lap. Place the nose cleaning probe in the nostril in such a way that the end of the short silicon tube with the collar should reach into the mucus. The mucus will appear in the probe within a few seconds. You should perform this operation on both nostrils.

WARNING: The probe with the silicon tube with the collar must not be pushed all the way into the nostril, because it may cause discomfort to the baby, or might injure the nostril.

ENGLISH

After successful nose cleaning you should cut the current off by turning it off, then unplugging the adapter from the socket. Pull the silicon tube attached to the pump off the suction stub. In case of fault or abnormal operation turn device off and unplug adapter from mains.

CLEANING THE NOSE PROBE:

Pull silicone tube off the nose probe. Wash both ends of the probe in a detergent solution, rinse thoroughly, then sterilise by boiling for 30 minutes.

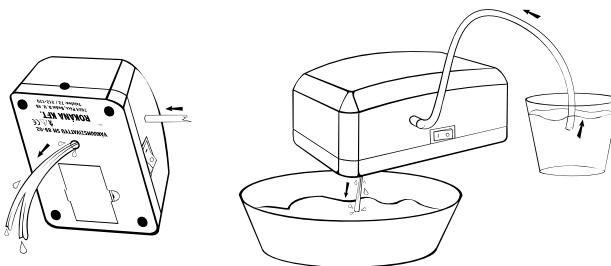
CLEANING THE VACUUM PUMP:

The outer casing of the vacuum pump should be cleaned with a cloth dampened with a detergent solution. Should milk inadvertently get into the pump, it should be washed thoroughly as seen in the illustration.

Order of cleaning:

1. Fill a glass with lukewarm water.
2. Place the silicone suction tube of the pump into the water
3. Hold the device above a bowl.
4. Turn on the device.

It may be necessary to repeat this operation more than once.

Number of MEEI qualification: E0495E113, HU-344
 ORKI qualification number: 393/1992
 Adequacy marking: 

B type patient part
 Class of protection against indirect contact: II

TECHNICAL DESCRIPTION:

Type: SM 89-02
 Size of vacuum pump: 115 x 60 x 60 mm
 Weight: 0,15 kg
 Nenndaten:
 Voltage:
 U1 230 V ~50 Hz Power consumption: 7 W
 U2 120 V ~60 Hz Power consumption: 6,5 W
 Power supply unit: AC 230 V/DC 3 V 500 mA
 AC 120 V/DC 3 V 500 mA
 Power supply unit made by: FUJIKON
 Model: 035-03-500

Continuous operation with intermittent duty
 Loading time: 3 minutes,
 Spacing time: 1 minute
 Degree of protection: IP20
 Vacuum: -0.4 bar
 Application temperature: between 0 and 40 °C
 Storage temperature: between 0 and 40 °C

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GUARANTEE

Error indication:
 Cause of error:
 Fault clearing: Service

GUARANTEE

Error indication:
 Cause of error:
 Fault clearing: Service

GUARANTEE

Error indication:
 Cause of error:
 Fault clearing: Service

GUARANTEE

Error indication:
 Cause of error:
 Fault clearing: Service

GUARANTEE

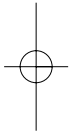
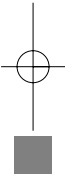
Error indication:
 Cause of error:
 Fault clearing: Service

GUARANTEE

Error indication:
 Cause of error:
 Fault clearing: Service

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Distributor:

ROKÁNA Kft.

H-7624 Pécs, Buday Dezső u. 48., Hungary

Tel./fax: +36-72/212-170

E-mail: rokanakft@t-online.hu

Web: www.rokana.hu